


IDI-9011 Drill Extension User's manual 正確安裝說明


Step 1

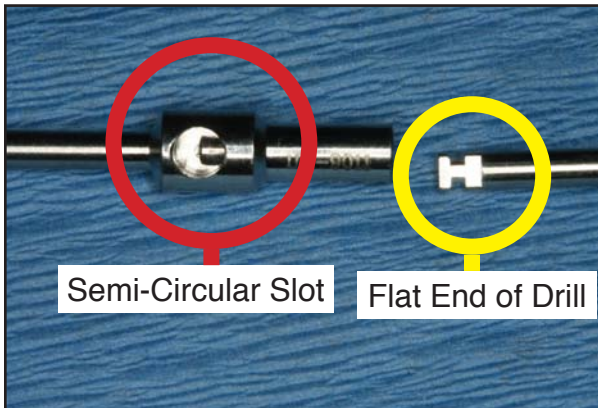

Put IDI-9011 Drill Extension's semi-circular slot and Drill's flat end facing yourself.

請將IDI-9011鑽頭延長上的圓孔及鑽頭尾端的平面端面對自己。


Step 2

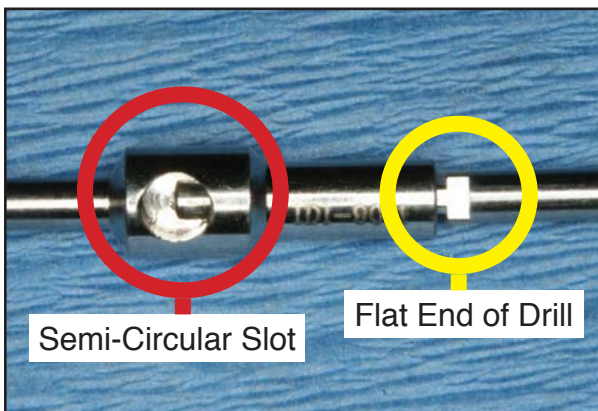

When seating drill into IDI-9011 Drill Extension, **DO NOT** twist nor spin both instruments, keep IDI-9011 Drill Extension's semi-circular slot and Drill's flat end facing yourself.

連接IDI-9011鑽頭延長及鑽頭時，請勿轉動。持續將IDI-9011鑽頭延長上的圓孔及鑽頭尾端的平面端面對自己。

Step 3

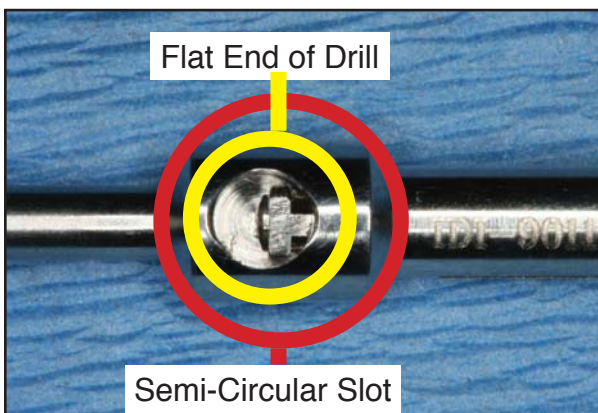

You can see the flat end of the drill seating on semi-circular slot of IDI-9011 Drill Extension via window. **(DO NOT twist the drill during and after seating.)**

連接完成後，經由鑽頭延長上的圓孔視窗，可以看到鑽頭尾端的平面端，如此才是正確安裝程序。（安裝過程及安裝完成後，請勿轉動鑽頭）